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## EN

*Instructions for use revised in March 2018.*

### GENERAL INFORMATION

ALICE pliers, MANOTTE 1.6TB pliers and HK2 pliers are **supplied non-sterile**. Qualified personnel will use them. Instruments must be used only for the surgical purpose and type for which they have been designed. Any breach of this rule cancels the guarantee:

- 5 years for instruments
- 1 year for parts in use such as jaws

If for any reason an instrument needs to be returned to us, it must be first decontaminated. AREX® reserves the right to request a decontamination proof.

### INDICATION FOR USE

- ALICE pliers have been designed to push/remove and cut K-wires from 0.8 mm up to 1.2 mm in diameter. **Do not use it as a wire bender.**
- MANOTTE 1.6TB pliers can bend and cut K-wires from 1 mm up to 1.6 mm in diameter.

They also crimp K-wire tip protection in place. MANOTTE 1.6TB are designed for LIGAMENTOTAXOR device settings.

- HK2 pliers can bend and cut 1.2 mm, 1.5 mm and 1.8 mm in diameter K-wires. They also crimp corresponding connectors. HK2 pliers are specifically designed to set HK2 system.

### STORING

ALICE, MANOTTE 1.6TB and HK2 pliers must be stored in a clean and dry environment, at room temperature.

Do not store instruments near potentially corrosive substances, particularly chlorine.

### CLEANING

It is recommended to clean new instruments before first sterilization and after each use. Always remove any oily residue on items before cleaning. Instruments made out of stainless steel, such as ALICE, MANOTTE 1.6TB and HK2 pliers, are designed to withstand intensive cleaning in sodium hydroxide solutions, notably used for elimination of TSE agent.

### Cleaning process

#### Pre-cleaning

No further pre-cleaning necessary. Cleaning should occur as soon as possible after device use in order to prevent the drying or fixation of soil.

#### Automated cleaning

The detergent used is enzymatic solution\*. Use cleaning program similar to the one described below:

1. 12 min pre-cleaning with cold tap water.
  2. Drying.
  3. 5 min cleaning at 55°C with 0,5% solution of the cleaning detergent\*.
  4. Drying.
  5. 3 min rinsing with cold demineralized water.
  6. Drying.
  7. 2 min final rinsing with cold demineralized water.
  8. Drying.
- \* The detergent manufacturer's recommendations will be followed.

### MAINTENANCE

After cleaning, apply a small dose of surgical lubricant oil on joints to optimize use of instruments.

After every cleaning, and drying, check on the product the following elements: dry state, cleanliness, proper functioning, and absence of deterioration.

### CHECKS AND TESTS WORKING ORDER

Check that joints move easily, without excessive looseness. Check on locking mechanisms (hook, clamping, ...). Inspect visually for signs of wear and tear. Cutting edges must be continuous and unbroken. Check gripper axis of pliers to detect any sign of distortion.

### PACKAGING

Individual: standard packaging material may be used. Make sure that the pouch is sufficiently large to house the instrument without stressing bag welding.

### STERILIZATION

ALICE, MANOTTE 1.6TB and HK2 pliers must be sterilised in pre-vacuum autoclave according to one of the following protocols: whether A or B.

### Protocol A

- 3 pre-vacuum phases
- Sterilization temp.: 132°C
- Holding time: 4 min
- Drying time: 20 min

### Protocol B

- 3 pre-vacuum phases
- Sterilization temp.: 134°C
- Holding time: 3 min
- Drying time: 20 min

Make sure that all inward and outward surfaces of the device are freely accessible during the sterilization.

Only cleaning and sterilization processes defined in this leaflet have been validated, however, the relevant Healthcare Unit may use their standard disinfection and sterilization protocol, provided the said protocol is validated for the disinfection and the sterilization of surgical instruments (for instance: sterilization in instrument holder - manual cleaning, other parameters of cleaning and sterilization).

### REUSE OF THE PRODUCTS

The instruments are reusable. However, the operating lifetime of the instruments is limited.

The instruments have been tested for 50 cycles of cleaning and sterilization. The operating lifetime in clinical practice depends on individual utilization and conditions of treatment specific to the Healthcare unit.

### RULES FOR APPROPRIATE USE OF INSTRUMENTS

- Do not use instruments still warm after sterilization cycle (risk of irreversible jamming).
- Do not allow an instrument to retain humidity on a laser marking. If this occurs a brown stain may appear, rub hard with a soft cloth.
- If instruments are broken or damaged, and cannot be repaired, waste disposal should be done according to the law.
- It is recommended to clean instruments as soon as possible after use.

### WARNING

- All instruments covered by this instruction for use must be used according to this IFU. Any information related to use of these instruments are available on AREX® web site: [www.arex.fr](http://www.arex.fr)
- Read all sections of this leaflet before use.
- Failure to follow good practices for cleaning and sterilizing surgical instruments may lead to infections or device malfunction.
- The use of a device for another task than that for which it has been designed may lead to device premature wear.
- Before use, inspect devices to ensure its condition and good functioning.
- Do not use instruments if they do not perform the intended function correctly or if they are damaged.

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