

ForgetMeNot - ref.: FMNY2B (yellow) or FMNB2B (blue)



Simple answer to a real issue*: digit tourniquet ForgetMeNot

Available in yellow or blue

Reusable

Sold in packs of 10

Sold no sterile

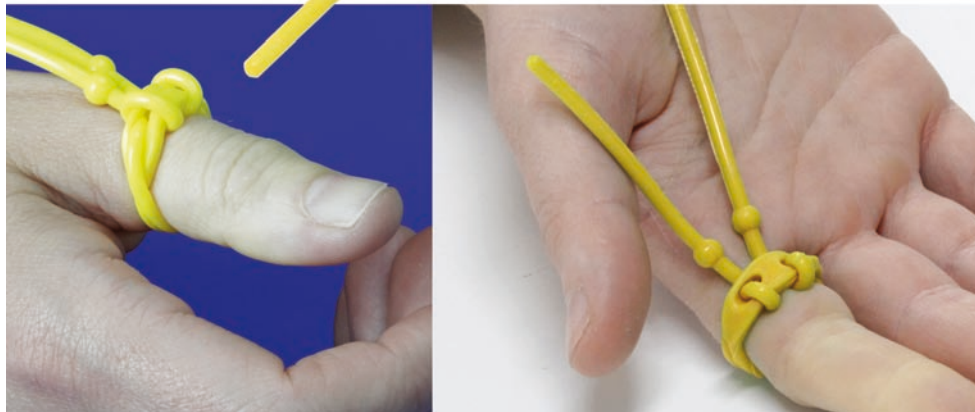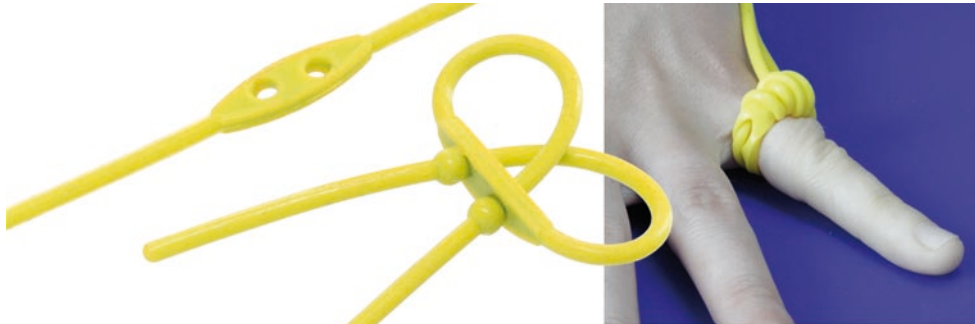

* Résultats fonctionnels à long terme d'ischémies digitales sous garrot: à propos de 2 observations.
SOFcot 2010 / Florence Mallard (Angers) Julien Richou, Dominique Le Nen



Quick video to present exsanguination

by Stéphanie Gouzou - Strasbourg University Hospital

CLICK HERE

<http://www.arex.fr/QRc.php?c=23>



Manufacturer:
AREX®
3 allée du Clos Tonnerre
91120 Palaiseau - FRANCE

www.arex.fr

ForgetMeNot

The only specifically designed-for-purpose, reusable finger tourniquet.

Reusable

One size-fits-all


Achieving a totally exsanguinated finger

Bright blue or yellow highly visible tails make it totally unforgettable

Removable without rolling it over the wound or any risk of cutting skin

Environmentally friendly since it is reusable

For optimal FMN use, follow the detailed instructions below.
Placement of the FMN tourniquet is simple, easy and repeatable.
It is used under local (LA) or locoregional (LRA) anaesthesia.

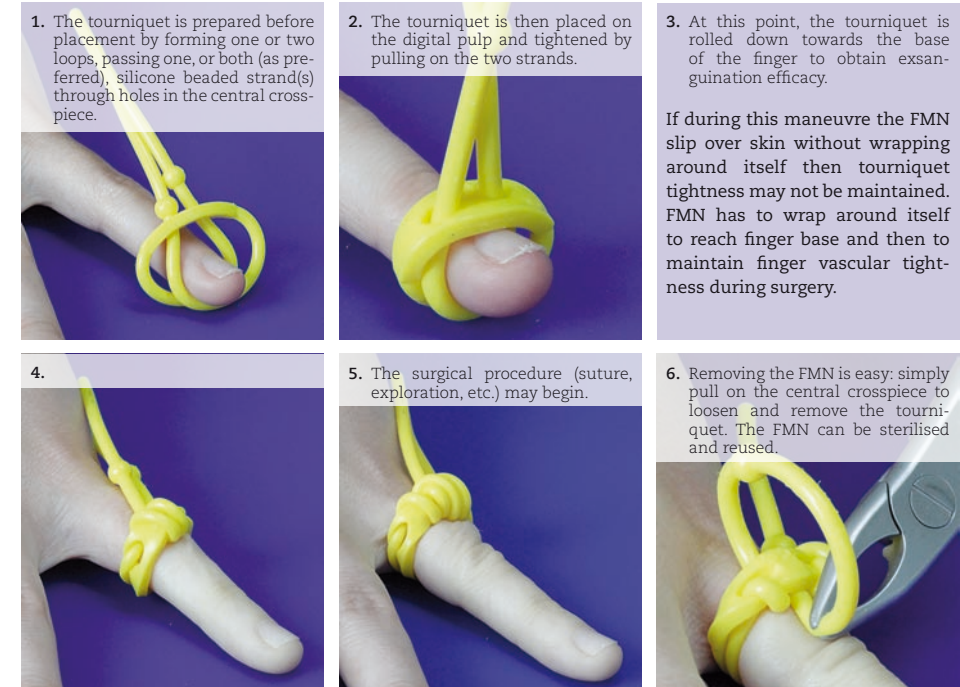

When a totally bloodless field is not required, the FMN can be used with a single loop: the arm is held above heart level for a few minutes and the FMN is applied directly to the base of the finger.

NB1 - Do not use ForgetMeNot on small fingers.

NB2 - Do not use ForgetMeNot in paediatric cases.

These contra-indication are related to FMN dimentional characteristics.

NB3 - Do not clamp ForgetMeNot with a metallic instrument to maintain this finger tourniquet, this may initiate small cuts into silicone resulting in potential secondary rupture.