ForgetMeNot

Garrot de doigt
Finger tourniquet
Finger-Tourniquet
Torniquete para dedo
Laccio emostatico per il dito
Garrote de dedo
Vingertourniquet

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EN

DEVICE AND INDICATION FOR USE

ForgetMeNot [FMN] is a reusable malleable silicone, device designed to be used alone as finger tourniquet in order to stop the blood circulation and exsanguinate the treated finger.

It is intended to be used on adult size hands.

These devices have been designed to facilitate hand surgery. These devices which are supplied non-sterile must undergo an automatic cleaning step followed by a steam sterilization step before use.

The steps for positioning ForgetMeNot on the patient are illustrated in annex.

WARNINGS

- Do not use the instrument when it comes out of the sterilization cycle and is still warm: there is a risk of premature wear.
- ForgetMeNot must be unfolded before cleaning and sterilization.
- Do not use the instruments if they are damaged.
- Do not leave the ForgetMeNot tourniquet in place for more than 20 minutes. The user must comply with local, national and international clinical protocols and guidelines. Exceeding this limit may lead to necrosis of the treated finger.
- Do not use the ForgetMeNot tourniquet in ischemic patients.
- Do not use the ForgetMeNot tourniquet on a child's finger or on small hands.
- Do not clamp the ForgetMeNot with a metal instrument; this can make small cuts in the silicone that may cause failure later on.
- Failure to follow good practices for the cleaning and sterilization of surgical instruments may lead to infections or device malfunction.
- Do not use ForgetMeNot for any use other than those defined by the manufacturer.

LIMITATIONS ON REPROCESSING

The instruments can be reused, however, their lifespan is limited. The lifespan of ForgetMeNot depends on the individual use (*wear, damage*).

The devices can be reused if the conditions of cleaning and sterilization provided by AREX® are followed. The mechanical strength of these devices has been confirmed by AREX® after the equivalent of 225 cleaning/sterilization cycles.

In order to avoid damaging of ForgetMeNot, it is forbidden to use solutions containing: soda and aromatic derivatives compounds (*Ether, Chloroform, Benzene, Xylene etc.*) or any other abrasive product when the cleaning and the decontamination these devices. In the case of suspected contamination by non-conventional transmissible agents (*NCTA: prion disease*), the devices cannot be reused and must be destroyed according to current regulations.

INSTRUCTIONS

1. Putting the new product into circulation (*out of the packaging*):

Remove the devices from their packaging.

It is recommended to clean the new device before the first sterilisation according to point 5.

2. Initial processing at point of use (after surgical use):

Cleaning must be carried out immediately after each use of the medical device to prevent dirt adhesion. Remove any excess dirt.

3. Containment and transport

The mode of transport must be secured, no additional requirements.

4. Preparation before automatic cleaning

It is recommended to clean new instruments before their initial sterilization and immediately after each use in order to avoid dirt adhesion.

ForgetMeNot will be cleaned like surgical instruments.

5. Automated cleaning

The used detergent is an enzyme-based or alkaline based. Use a cleaning program similar to this one described below:

1. Pre-cleaning
Tap water / Cold water / 2 min.
2. Drying
3. Cleaning
Tap water + Medizyme enzymatic ou Mediclean forte, alkaline (Neodisher) 0.5% / 55°C / 5 min.*
4. Drying
5. Intermediate rinsing
Demineralized water / Cold water / 3 min.
6. Drying
7. Final rinse
Demineralized water / Cold water / 2 min.
8. Drying

** Follow the detergent manufacturer's instruction.*

Note: A high temperature disinfection step can take place after the final rinse to prevent the fixation of any remaining proteins.

It is recommended to use a washer/disinfector conforms to ISO 15883.

2 automatic cleaning cycles can be performed prior to sterilization if necessary.

After cleaning, always check that the devices are dry, clean, in good working order and absence of deterioration.

6. Automated drying

Dry ForgetMeNot according to the drying cycle of the washer. If necessary, additional manual drying can be performed using lint free towel.

7. Maintenance

Non-applicable.

8. Controls and functional tests

After each cleaning and drying, check the dryness, cleanliness, proper functioning and absence of deterioration of the devices.

Before each use, carry out a visual inspection of the device to identify any damage wear and/or signs of degradation: crack.

9. Packaging

Standard individual packaging compatible with steam sterilization method must be used (*i.e. sealed bags*).

10. Sterilization

ForgetMeNot must be unfolded before sterilization. ForgetMeNot is sterilized in a vacuum autoclave according to one of the following protocols:

Protocol A

- 3 pre-vacuum phases
- Sterilization temperature: 132°C
- Holding time: 4 minutes
- Drying time: 20 minutes

Protocol B

- 3 pre-vacuum phases
- Sterilization temperature: 134°C
- Holding time: 3 minutes
- Drying time: 20 minutes

The instructions of the autoclave's manufacturer must be followed.

Ensure that the surfaces of the device are not covered during sterilisation.

Only the cleaning and sterilization procedures defined in this IFU have been validated. However, the healthcare facility can use its standard disinfection and sterilization protocol, as long as this protocol has been validated for the disinfection and sterilization of surgical instruments (*e.g. sterilization in instrument holders, manual cleaning, other cleaning and sterilization parameters*).

STORAGE CONDITIONS

ForgetMeNot must be stored in a dry and clean and dry environment, at room temperature.

DISPOSAL

Any instrument that is broken, damaged or at the end of its life must not be used. It must be destroyed like any medical waste according to current regulations.

ADDITIONAL INSTRUCTIONS

1. General Information

ForgetMeNot is provided non-sterile.

Devices must be used by surgeon and qualified personnel. Instruments must be used only for the surgical purpose and type for which they have been designed (*see indications*). Any failure to respect these rules cancels the 1 year guarantee.

If for any reason, some instrument needs to be returned to us, it must be first decontaminated. AREX® reserves the right to request a decontamination proof.

2. Precautions for use

- ForgetMeNot must be used in accordance to this Instruction for Use. All the information related to use of these instruments is available on the AREX® website: www.arex.fr
- Read all the sections of this IFU before use.
- The instruments should be cleaned immediately after use.
- Before use, inspect devices to verify its condition and proper functioning.

3. Incident declaration

Any serious incident that has occurred in relation to ForgetMeNot should be reported to AREX®, and the competent authority of the Member State in which the user and/or patient is established.

Reminder: A serious incident or risk of serious incident is any incident or risk of incident involving a medical device which led or might have led to death or serious deterioration in state of health of patient, end-user and third party.

These instructions have been validated by the medical device's manufacturer as being appropriate for preparing a medical device for reuse. The department is still responsible for ensuring the sterilization process achieves the intended result by involving the sterilization facility's staff, materials and equipment. This generally requires routine verification and control of the procedures. Similarly, any deviation from the instructions provided should be evaluated to determine the effectiveness and possible adverse consequences.

IFU Revision date: August 2020

The link to the explicative video is available
via the following QR-code:

<http://www.arex.fr/QRc.php?c=23>

The technique sheet and the explanatory video
are also available on the AREX[®] website.

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