CHIROBLOC®
ForgetMeNot®

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GENERAL INFORMATION

Family of products: CHIROBLOC®, CHIROBLOC® BALLAST and ForgetMeNot® are **supplied NON-STERILE**.

Qualified personnel must use them. Instruments must be used only for the surgical purpose and type for which they have been designed. Any breach of this rule cancels the guarantee:

CHIROBLOC®: 1 year

CHIROBLOC® BALLAST: 5 years

If for any reason an instrument needs to be returned to us, it must be first decontaminated. AREX® reserves the right to request this decontamination proof.

INDICATION FOR USE

- CHIROBLOC® (CBM, CBP and CHIROBLOC® 3) have been designed to maintain the hand in the desired position during surgical operation.
- The CHIROBLOC® BALLAST is an accessory which enable to ballast the device in order to tilt the CHIROBLOC® in the desired position.
- ForgetMeNot® is designed to be used as a tourniquet for the finger it can be used for toe too. It is also an accessory for CHIROBLOC® 3.
- CHIROBLOC® (CBM, CBP and CHIROBLOC® 3) is compatible with MRI (Magnetic Resonance Imaging).

STORING

CHIROBLOC®, ForgetMeNot® and CHIROBLOC® BALLAST products must be stored in a clean and dry environment.

Do not store instrument, CHIROBLOC® and ForgetMeNot® close to potentially corrosive products (*chlorine*).

CLEANING

It is recommended to clean new instruments before first sterilization and after each use.

• CHIROBLOC® BALLAST

CHIROBLOC® BALLAST is made out of stainless steel, so it can withstand intensive cleaning with soda based solution. It will be cleaned as instruments made out of stainless steel.

Pre-cleaning

No further pre-cleaning necessary. Cleaning should occur as soon as possible after device use in order to prevent the corrosion of the instruments and the drying or fixation of soil.

• CHIROBLOC® (CBM, CBP and CHIROBLOC® 3)

Can undergo automated cleaning or manual cleaning.

Pre-cleaning

No further pre-cleaning necessary. Cleaning should occur as soon as possible after device use in order to prevent the drying or fixation of soil.

Automated cleaning

Detergent used is enzymatic solution*.

Use cleaning program similar to the one described below:

1. 2 min pre-cleaning with cold tap water.
2. Drying.
3. 5 min cleaning at 55°C with 0.5% solution of the cleaning detergent*.
4. Drying.
5. 3 min rinsing with cold demineralized water.

6. Drying.
 7. 2 min final rinsing with cold demineralized water.
 8. Drying.
- *The detergent manufacturer's recommendations will be followed.*

Manual cleaning

Enzymatic detergent, brush (do not use metal brushes, scouring, and abrasive pads), running water. Method:

1. Rinse excess dirt from instrument (*immerse in detergent for 20 min*).
2. Rinse in clean water for at least 3 min.
3. Rinse instruments with purified water for at least 3 min or until any visible blood or dirt is eliminated.
4. Remove excess moisture from the instrument with a clean, absorbent and fluff-free cloth.

• **ForgetMeNot®**

CHIROBLOC® cleaning method will apply.

PACKAGING

Individual: standard packaging material may be used. Make sure that the pouch is sufficiently large to house the instrument without stressing bag welding. This is why CHIROBLOC® BALLAST will be packed in a separate pouch.

STERILIZATION

Caution: CHIROBLOC® must not be folded before sterilization, this would reduce its product life.

CHIROBLOC®, ForgetMeNot® and CHIROBLOC® BALLAST are sterilised in pre-vacuum autoclave according to following protocol: whether A or B.

Protocol A

- 3 pre-vacuum phases.
- Sterilization temperature: 132°C
- Holding time: 4 min
- Drying time: 20 min

Protocol B

- 3 pre-vacuum phases.
- Sterilization temperature: 134°C
- Holding time: 3 min
- Drying time: 20 min

Only cleaning and sterilization processes defined in this leaflet have been validated, however, the relevant Healthcare Unit may use their standard disinfection and sterilization protocol, provided the said protocol is validated for the disinfection and the sterilization of surgical instruments (*for instance: sterilization in instrument holder – manual cleaning, other parameters of cleaning and sterilization*).

RE-USE OF THE INSTRUMENTS

Unless labelled for single use the instrument could be re-used. However, surgical instruments have a limited lifespan.

RULES FOR APPROPRIATE USE OF INSTRUMENTS

- Do not use instruments still warm after sterilization cycle (*risk of premature wear*).
- Do not allow an instrument to retain humidity on a laser marking. If this occurs a brown stain may appear, rub hard with a soft cloth.
- Do not fold CHIROBLOC® before sterilization, this would reduce drastically its product life.
- If instruments are broken or damaged, and cannot be repaired, waste disposal should be done according to the law.

- CHIROBLOC® BALLAST should be cleaned separately from CHIROBLOC® hand.
- It is recommended to clean instruments as soon as possible after use.

WARNING

- All instruments covered by this instruction for use must be used according to this IFU, any information related to use of these instruments is available on AREX® web site: www.arex.fr
- Read all sections of this leaflet before use.
- Failure to follow good practices for cleaning and sterilizing surgical instruments may lead to infections or device malfunction.
- The use of a device for another task than that for which it has been designed may lead to device premature wear.
- Before use, inspect devices to ensure its condition and good functioning.
- Do not use instruments if they do not perform the intended function correctly or if they are damaged example: If a defect is found at the plug of the product is non-compliant and will not be used.
- Only cleaning and sterilization processes defined in this leaflet have been validated.
- Do not let in place more than 40 min the finger tourniquet ForgetMeNot® or do not use on ischaemic patients. Conform to local, national and international clinical protocols and guidelines. Exceeding this limit may result in necrosis of the treated finger.
- Do not use tourniquet, ForgetMeNot® on children's fingers or on small hands.
- Do not clamp ForgetMeNot® with a metallic instrument to maintain this finger tourniquet, this may initiate small cuts into silicone resulting in potential secondary rupture.

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